

Introducing

On-Call[®] Stealth[™] Lancing Device



**Bright Yellow Release Button
Indicates Ready to Use**


More Comfort for your Patients

9 out of 10 patients think Stealth is less painful than
their current lancing device*




Back Detail

- Includes clear cap for alternate site testing
- 11 depth settings provide the user with ultimate control over penetration depth for maximum comfort
- For use with universal lancets
- Designed for easy gripping and smooth release
- Lancet ejector to avoid accidental needle sticks
- Durable and long lasting for repeated use
- Easy draw with light spring mechanism

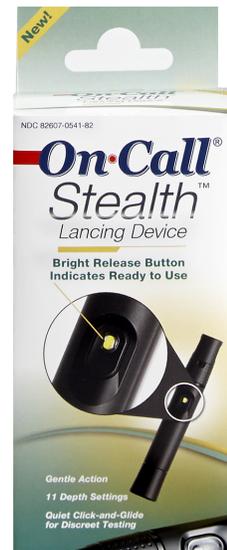

Lancing Device Pain Evaluation Study

	1 = No Pain	2 = Slightly Painful	3 = Somewhat Painful	4 = Painful	5 = Very Painful
Accu-Chek® FastClix	54	34	7	4	1
On Call® Stealth™	51	36	10	2	1
OneTouch Delica™	34	44	17	4	1
ReliOn® (CanAm)	34	42	16	7	1
Nipro TRUEdraw™	9	45	20	17	9

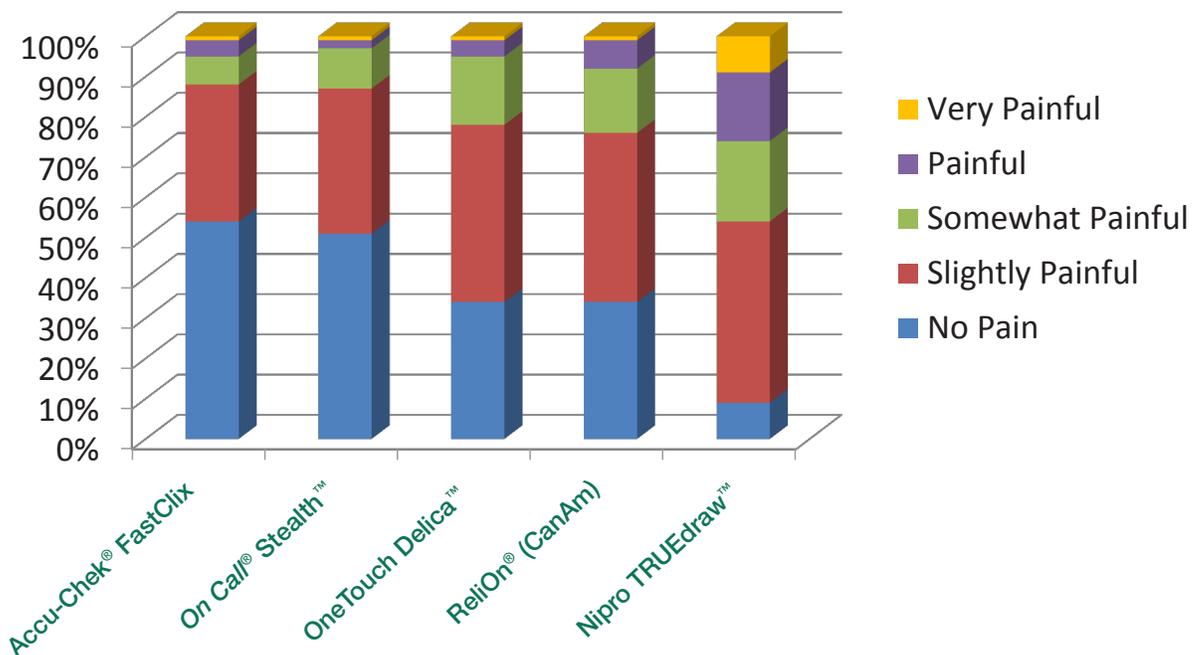

Combined Response (2 responses per Subject)

On Call® Stealth™ vs. Accu-Chek® FastClix

28 Responses ranked On Call® Stealth™ as less painful
 26 Responses ranked Accu-Chek™ as less painful
 46 Responses Ranked them the same

No Statistically Different?

On Call® Stealth™ vs. OneTouch Delica™

40 Responses ranked On Call® Stealth™ as less painful
 20 Responses ranked OneTouch Delica™ as less painful
 40 Responses Ranked them the same

Yes (p<0.05)

Statistically Different?

1. 50 blindfolded subjects x 2 responses per subject

2. Lancing depth for each device normalized to the minimum depth required for a 0.4 µL sample from each subject

All trademarks are the property of their respective owners.

ACON Laboratories does not guarantee the accuracy or completeness of the information and data, makes no representations of warranties, express or implied, and assumes no liability associated with use or reliance thereof. For informational purposes only.