
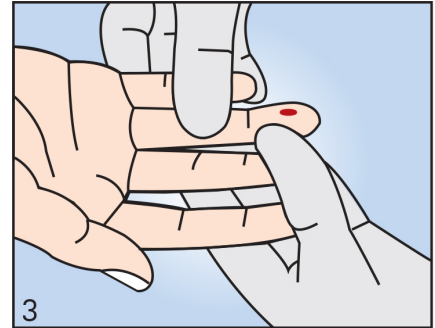
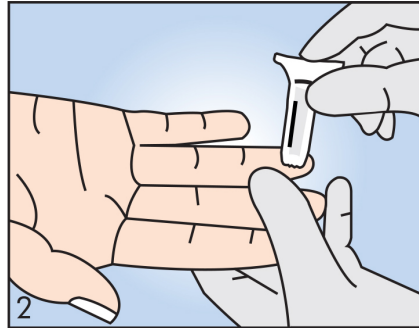


Instructions for Use

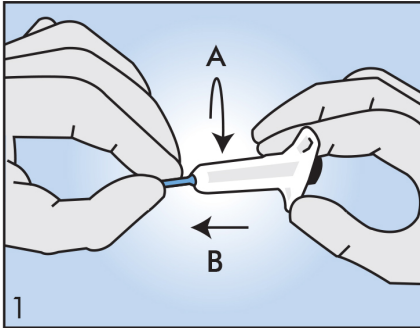

English

1. Twist colored cap 360° and pull to remove.

2. Press firmly against the puncture site and push the button to lance.

3. Collect the required blood volume and discard the used lancet into an appropriate sharps container in accordance with facility guidelines.

Warning: Do not use if protective cap has been removed from the lancet.

Español

1. Gire la tapa de color a 360 ° y hale para remover.

2. Presione firmemente contra el punto de punción y pulse el botón para punzar.

3. Colecte la cantidad de sangre requerida y desechar la lanceta usada en un recipiente previsto, de acuerdo con las normas de su institución.

Advertencia: No utilizar si la tapa protectora ha sido retirada de la lanceta.

Français

1. Tournez l'embout de couleur à 360° et tirez pour le retirer.

2. Appuyez fermement contre le site de ponction et poussez le bouton-poussoir pour activer la lancette.

3. Recueillir le volume de sang nécessaire et jeter la lancette usagée dans un contenant approprié pour objets tranchants conformément aux directives de l'établissement.

Avertissement: Ne pas utiliser si l'embout protecteur a été retiré de la lancette.