


# OptiLance™

Push-Button Safety Lancets

## It's what's inside that matters

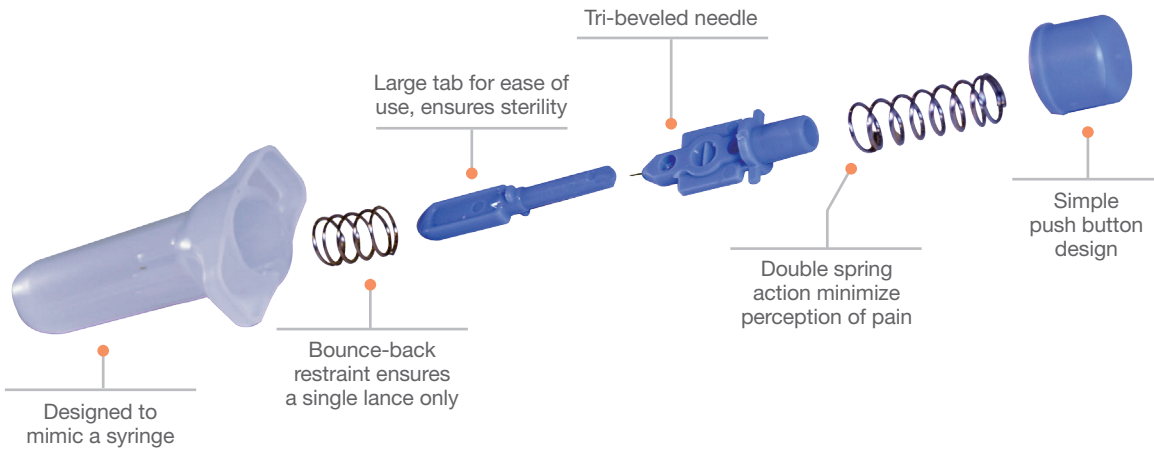

Unique cupping ridges designed to divert attention **from pain and help grip the skin for easier, steadier and more comfortable sampling.**




**ACON®**

# Ergonomically designed to provide you with optimal safety and comfort

## DIRECTIONS FOR USE

### Step 1 - Remove Cap


Grasp the outer white edge of the OptiLance lancet when twisting off the protective cap.

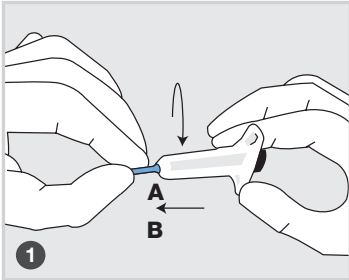

Twist cap 360° until loose to easily remove the protective cap.


### Step 2 - Position Lancet

Press lancet against side of fingertip and press button firmly.

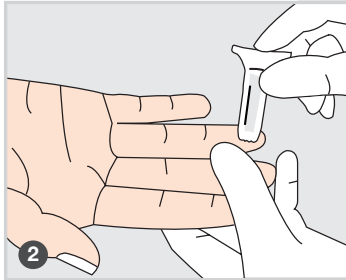




### Step 3 - Dispose

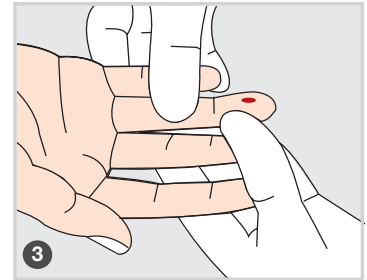After use, dispose lancets in appropriate sharps container.



**Note:** Do not put your thumb on the color button while removing protective cap. This may cause a misfire.

## ORDERING INFORMATION

OptiLance	Size	Blood Volume Suggested Application	Catalog Number	
			100ct	200ct
	<p>●</p> <p><b>28G Needle</b> <b>1.2mm Depth</b></p>	1-2 µl for thin or normal skin BG testing	761131	761132
	<p>●</p> <p><b>25G Needle</b> <b>1.6mm Depth</b></p>	1-2 µl for thick skin BG testing, 5-10 µl for normal skin POC testing	761129	761130
	<p>●</p> <p><b>21G Needle</b> <b>2.0mm Depth</b></p>	30-40 µl for POC testing	761127	761128