

On·Call[®] Pro

Blood Glucose Monitoring System

Cleaning and Disinfecting the On Call[®] Pro Meter


The cleaning and disinfection procedures described below should be followed in order to minimize risk of transmitting blood-borne pathogens.

Clean and disinfect the meter after each use. Standard precautions, your institution's guidelines, and the manufacturer's procedures described below must be followed when using the On Call[®] Pro Blood Glucose Monitoring System.

Visit www.oncallmeters.com for the most up-to-date information on approved cleaning and disinfecting procedures and disinfection materials for the On Call[®] Pro Blood Glucose Monitoring system.


Cleaning Procedure

1. Wear appropriate protective gear as recommended by your institution.
2. Pre-clean the meter with a DisCide Ultra Disinfecting Towelette (EPA Registration No. 10492-4) or a PDI Super Sani-Cloth Germicidal Disposable Wipe (EPA Registration No. 9480-4) by wiping the entire meter surface to remove any stains or debris. Avoid allowing any liquids to enter the strip port or data port.
3. Dispose of the used towelette.

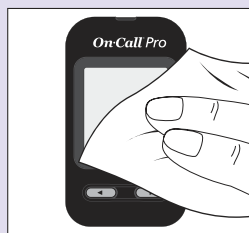

Disinfection Procedure

4. Use another DisCide Ultra Disinfecting Towelette or PDI Super Sani-Cloth Germicidal Disposable Wipe to disinfect the meter by wiping the entire surface thoroughly. Avoid allowing any liquids to enter the strip port or data port.
5. Dispose of the used towelette.
6. Allow the surface of the meter to remain wet according to the contact times listed below.

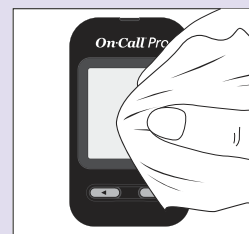

Disinfection Wipe	EPA Registration No.	Contact Time
DisCide Ultra Disinfecting Towelette	10492-4	1 minute
PDI Super Sani-Cloth Germicidal Disposable Wipe	9480-4	2 minutes

PLEASE NOTE:

- Only one of the two disinfectants listed should be used on the device for the lifespan of the device. The effect of using more than one disinfectants interchangeably has not been evaluated
- Avoid inserting the disinfecting wipe inside of the strip port or data port when performing pre-cleaning or disinfection
- DO NOT use any product containing bleach on the meter. This will void the warranty and may cause damage to the meter.
- Be sure to pre-clean and disinfect the meter after each use. This is important to potentially prevent the transmission of infectious diseases
- A total of 10,950 cleaning and disinfection cycles have been validated to represent cleaning and disinfection after each use over a 3-year lifespan.



©2016 ACON Laboratories, Inc. San Diego, CA 92121
PN 2160103002 8/16