

On·Call® Plus II

Blood Glucose Monitoring System



Improving for Accuracy and Convenience



- Accurate Results with Advanced Features
- User-Friendly Features for Better Diabetes Care
- Smart and Powerful Data Management




ACON®
Diabetes Care

Accurate Results with Advanced Features


- Accurate results in just 5 seconds
- Requires a small sample of only 0.5 µL
- Clinically proven accuracy within +/-15% that can comply with new ISO15197

Consensus Error Grid - On Call® Plus II System
Fingertip by Technician - Strip Lot K130321A-1

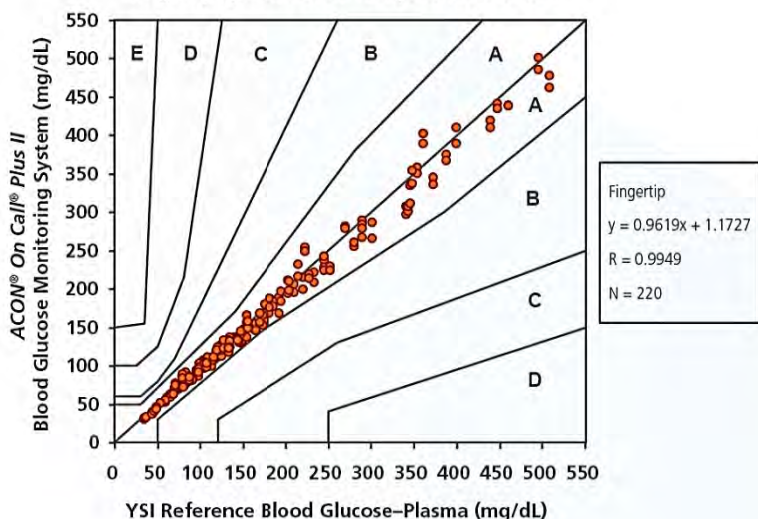


On Call® Plus II with Fingertip Sample Results (vs. Plasma YSI with Fingertip Sample)			
System Accuracy Results for Glucose Concentration > 100mg/dL			
Within ± 5%	Within ± 10%	Within ± 15%	Within ± 20%
94/154	135/154	154/154	154/154
(61.0%)	(87.7%)	(100%)	(100%)
System Accuracy Results for Glucose Concentration < 100mg/dL			
Within ± 5mg/dL	Within ± 10mg/dL	Within ± 15mg/dL	
44/66	64/66	66/66	
(66.7%)	(97.0%)	(100%)	
System Accuracy Results for both Glucose Concentration > 100mg/dL and < 100mg/dL			
Within ± 15% or ± 15mg/dL			
220/220 (100%)			

Consensus Error Grid - On Call® Plus II System
Palm by Technician - Strip Lot K130321A-1

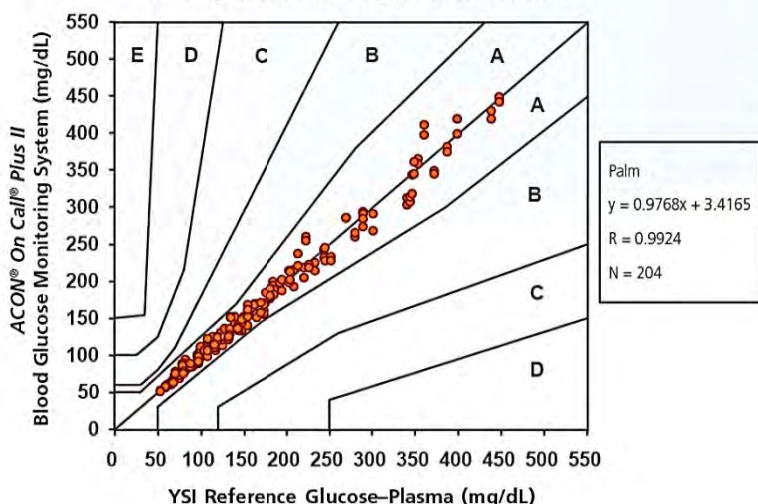

Consensus Error Grid - On Call® Plus II System
Forearm by Technician - Strip Lot K130321A-1

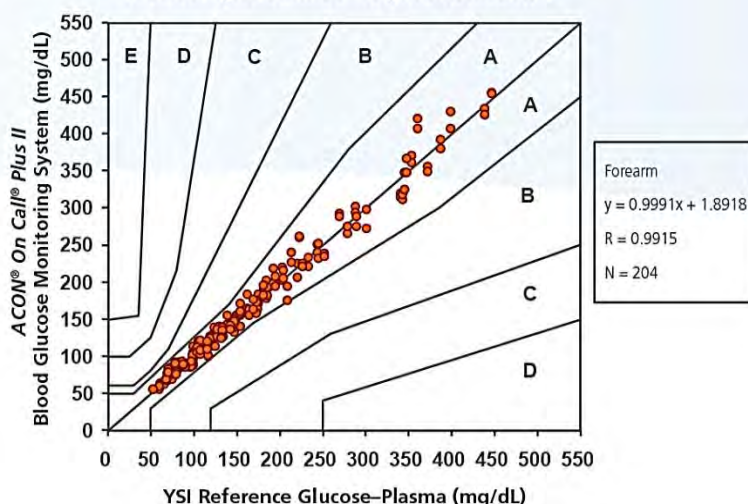

On·Call® Plus II

Blood Glucose Monitoring System

User-Friendly Features for Better Diabetes Care

- Uses GOD enzyme to avoid the interference from maltose, galactose, xylose and lactose
- Easy two-step testing




- Individually foil wrapped test strips available for added accuracy and safety



- On Call® Plus II strip can be used with On Call® Plus meter with 10 seconds testing time and 0.5 µL sample volume

Smart and Powerful Data Management

- 300 test memory with date and time
- 7, 14, and 30-day averaging allows users to quickly identify blood glucose trends
- Easy PC connection by USB cable
- On Call® Diabetes Management Software allowing for better data analysis

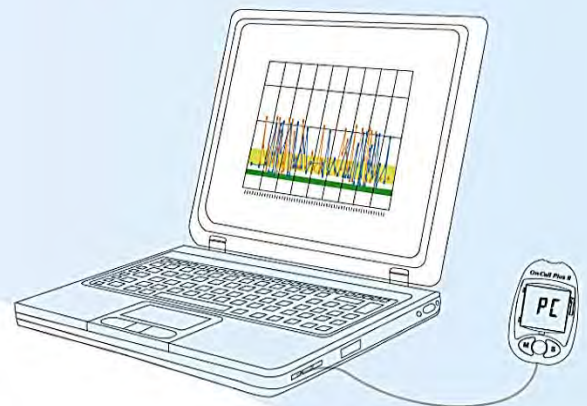

ACON®
Diabetes Care

On-Call® Plus II

Blood Glucose Monitoring System

Specifications

Feature	Specification
Measurement Range	1.1 - 33.3 mmol/L (20 - 600 mg/dL)
Result Calibration	Plasma-equivalent
Sample	Fresh capillary whole blood
Minimum Sample Size	0.5 µL
Test Time	5 seconds
Hematocrit Range	30 - 55%
Power Source	One CR 2032 3.0V coin cell battery
Battery Life	12 months or approximately 1,000 tests
Glucose Units of Measure	The meter is preset to either millimoles per liter (mmol/L) or milligrams per deciliter (mg/dL) depending on the standard of the country
Memory	Up to 300 records with time and date
Automatic Shutoff	2 minutes after last action
Meter Size	85 mm x 54 mm x 20.5 mm
Display Size	35 mm x 32.5 mm
Weight	Approximately 49.5 g (with battery installed)
Operating Temperature	5 - 45 °C (41 - 113 °F)
Operating Relative Humidity	10% - 90% (non-condensing)
Data Port	USB

Catalog

Product Name	Catalog No.	Contents
<i>On Call® Plus II</i> Blood Glucose Monitoring System	G113-10C ✓	1 Meter 10 Test Strips 1 Code Chip 1 Lancing Device 1 Control Solution 1 1 Manual 1 Carrying Case 1 Warranty Card 1 Quick Reference Guide 10 Lancets 1 Clear Cap (for testing on forearm and palm)
<i>On Call® Plus II</i> Blood Glucose Meter	G113-11C ✓	1 Meter 1 Control Solution 1 1 Carrying Case 1 Quick Reference Guide 1 Manual 1 Warranty Card
	G113-12C ✓	1 Meter 10 Lancets 1 Lancing Device 1 Control Solution 1 1 Manual 1 Warranty Card 1 Quick Reference Guide 1 Carrying Case 1 Clear Cap (for testing on forearm and palm)
<i>On Call® Plus II</i> Test Strips	G133-10C ✓	25 Test Strips (25/vial) 1 Code Chip 1 Package Insert
		50 Test Strips (25/vial) 1 Code Chip 1 Package Insert
		50 Test Strips (50/vial) 1 Code Chip 1 Package Insert
		100 Test Strips (25/vial) 2 Code Chips 1 Package Insert
	G133-11C ✓	25 Test Strips (Individually Foil Wrapped) 1 Code Chip 1 Package Insert
<i>On Call® Plus II</i> Glucose Control Solution	G123-12C ✓	1 Control Solution 0 1 Control Solution 1 1 Control Solution 2 1 Package Insert
<i>On Call®</i> Lancets	G124-10A ✓	100 Lancets (25/bag)
<i>On Call®</i> Lancing Device	G124-11A ✓	1 Lancing Device 1 Package Insert
<i>On Call® Diabetes</i> Management Software Kit	G124-13A	1 USB Data Transfer Cable 1 Installation Disk

✓ CE Marked for sale in the European Community  0123

