

On·Call[®] Extra


Blood Glucose Monitoring System

No Coding

NEW
ISO
15197

Extra Accuracy, Extra Control

- Reliable testing with clinically proven accuracy
- Fast 4 second testing
- Small 0.4 μ L sample

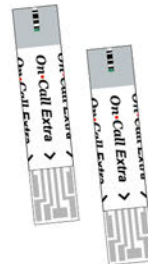


ACON[®]
Diabetes Care

Advanced Biosensor Technology for Fast and Accurate Glucose Testing

- Auto calibration circuitry inside every strip
- Accurate results that can comply with EN ISO15197:2015
- Just 4 seconds and only 0.4 µL blood sample



Consensus Error Grid - On Call[®] Extra System
Fingertip by Technician - Strip Lot 1190001, 1190002, 1190003

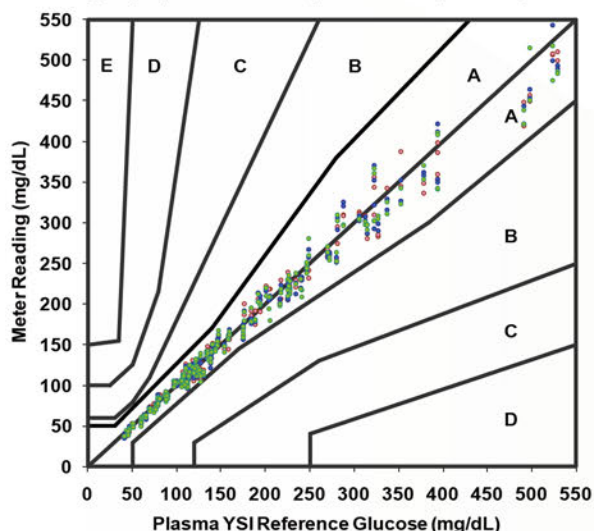


System Accuracy Results for Glucose Concentration ≥ 100 mg/dL		
Within ± 5 %	Within ± 10 %	Within ± 15 %
256 / 492 (52.0 %)	431 / 492 (87.6 %)	490 / 492 (99.6 %)


System Accuracy Results for Glucose Concentration < 100 mg/dL		
Within ± 5 mg/dL	Within ± 10 mg/dL	Within ± 15 mg/dL
132 / 174 (75.9 %)	174 / 174 (100 %)	174 / 174 (100 %)

System Accuracy Results for Glucose Concentrations Between 41 mg/dL and 574 mg/dL	
Within ± 15% or ± 15 mg/dL	
664 / 666 (99.7%)	

Consensus Error Grid - On Call[®] Extra System
Forearm by Technician - Strip Lot 1190001, 1190002, 1190003

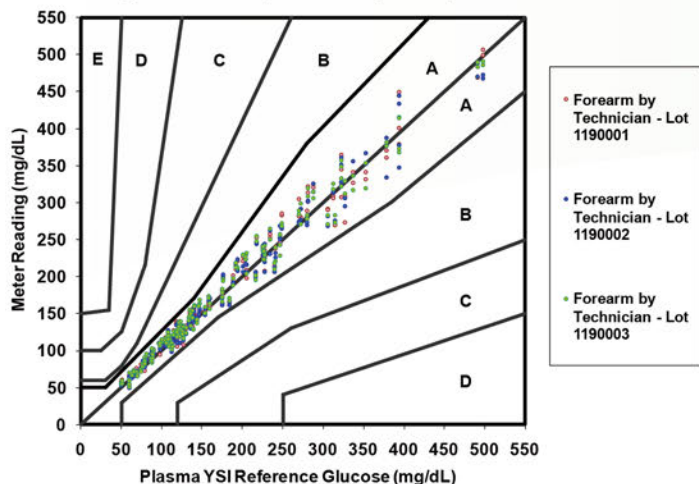

Consensus Error Grid - On Call[®] Extra System
Palm by Technician - Strip Lot 1190001, 1190002, 1190003

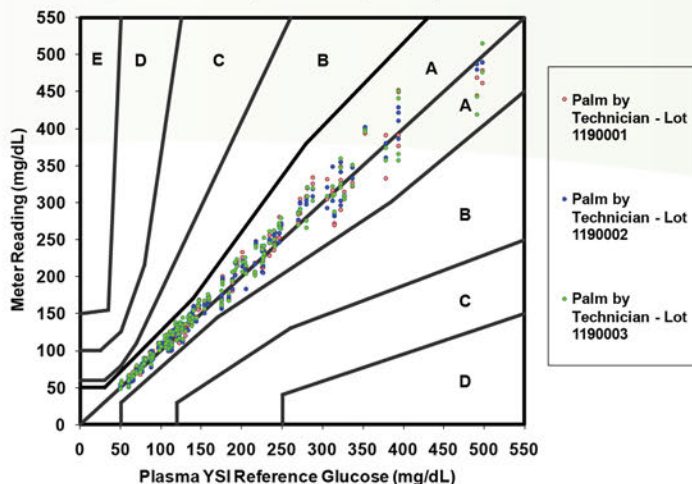




User-Friendly Features for Easier Diabetes Management

- Audible fill detection



- Strip ejector eliminates contact with used strips

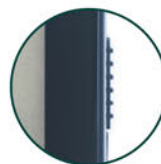

- Re-apply a second drop of blood within 3 seconds to avoid wasted strips



- Large display with bold numbers for better visibility




- Set up to 5 daily testing reminders



Smart and Powerful Data Management

- 300 test memory with date and time
- 7, 14, 30, 60 and 90 day averaging for Pre-meal, Post-meal and all test results
- Test results can be automatically marked as Pre-meal or Post-meal based on time of day
- Easy USB connection to *On Call*[®] Diabetes Management Software allows better data analysis
- Hypo, hyper and ketone warnings alert users to take necessary actions

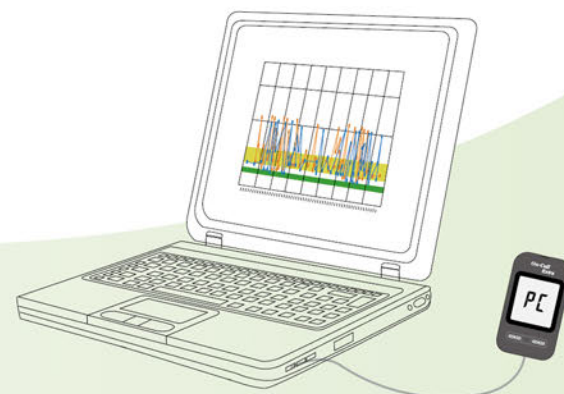

The *On Call*[®] Diabetes Management Software is available online for download. For more information, please contact your local distributor.



Specifications

Feature	Specification	
Technology	Biosensor/Electrochemical, Glucose oxidase (GOD)	
Calibration Method	No coding	
Test Time	4 seconds	
Sample Size	0.4 µL	
Sample Type	Fresh whole blood (Capillary)	
Test Sites	Fingertip, forearm, palm	
Hematocrit Range	25 - 60%	
Glucose Test Range	10 - 600 mg/dL (0.6 - 33.3 mmol/L)	
Memory Storage	300 results with date and time	
Test Averaging	7, 14, 30, 60, 90-day averages	
Data transfer	USB	
Control Solution	3 levels	
Audio Feature	Optional beep for sample detection, error messages	
Automatic Shutoff	2 minutes after last action	
Battery	One (1) preinstalled 3.0V CR 2032 lithium coin cell battery	
Battery Life	1,000 measurements	
Operating Conditions	41 – 113 °F (5 – 45 °C) and 10 – 90% relative humidity	
Strip Shelf Life	24 months	
Open Vial Expiration	6 months	
Diabetes Management Software	Available	
Other Functions	Insufficient sample volume detection	Ketone alert
	2 nd sample application allowed	Five (5) customizable test reminders
	Meal markers – before & after meals	Test strip ejector
	Hypo and hyper warnings	

All On Call[®] Extra Products are US FDA 510(k) cleared, CLIA Waived and CE Marked

Catalog

Product Name	Catalog No.	Contents
On Call [®] Extra Blood Glucose Monitoring System	√ G115-10H	1 Meter 10 Test Strips 1 Lancing Device 1 Control Solution 1 1 Manual 1 Carrying Case 1 Warranty Card 1 Quick Reference Guide 10 Lancets 1 Clear Cap (for testing on forearm and palm)
On Call [®] Extra Blood Glucose Meter	√ G115-11H	1 Meter 1 Lancing Device 1 Carrying Case 1 Manual 1 Warranty Card 1 Quick Reference Guide 1 Clear Cap (for testing on forearm and palm)
	√ G115-12H	1 Meter 1 Lancing Device 1 Carrying Case 1 Manual 1 Warranty Card 1 Quick Reference Guide 10 Lancets 1 Control Solution 1 1 Clear Cap (for testing on forearm and palm)
	√ G115-13H	1 Meter 1 Carrying Case 1 Manual 1 Warranty Card 1 Quick Reference Guide 1 Control Solution 1
On Call [®] Extra Blood Glucose Test Strips	√ G135-10H	25 Test Strips (25/vial) 1 Package Insert
		50 Test Strips (25/vial) 1 Package Insert
		50 Test Strips (50/vial) 1 Package Insert
		100 Test Strips (25/vial) 1 Package Insert
		100 Test Strips (50/vial) 1 Package Insert
	√ G135-11H	25 Test Strips (Individually foil wrapped) 1 Package Insert 50 Test Strips (Individually foil wrapped) 1 Package Insert
On Call [®] Extra Glucose Control Solution	√ G125-12H	1 Control Solution 0 1 Control Solution 1 1 Control Solution 2 1 Package Insert
On Call [®] Lancets	√ G124-10A	100 Lancets (25/bag)
On Call [®] Lancing Device	√ G124-11A	1 Lancing Device 1 Packaging Insert
On Call [®] Diabetes Management Software Kit	G124-13A	1 USB Date Transfer Cable 1 Installation Disk

√ CE Marked for sale in the European Community

