



## Results Chart and Interpretation

Color Chart Reading	Reagent	Results			
	Leukocytes (LEU)	- or ±	- or ±	+, ++, or +++	+, ++, or +++
	Nitrite (NIT)	-	+	+	-
<b>Summary of Results</b>		Leukocytes: Negative Nitrite: Negative	Leukocytes: Negative Nitrite: Positive	Leukocytes: Positive Nitrite: Positive	Leukocytes: Positive Nitrite: Negative
<b>Recommendations</b>		Results suggest no sign of UTI. When trace (±) LEU occurs or if you have symptoms, test again the following day with your first morning urine using a new strip. If you get another trace (±) LEU or if you still have symptoms, consult your physician.	Results suggest a sign of UTI. Consult your physician immediately.	Results suggest a sign of UTI. Consult your physician immediately.	Results suggest a sign of UTI. Repeat the test next time you urinate using a new strip. Make sure to wash the genital area first. If LEU is still positive, consult your physician.



ACON Laboratories, Inc. 10125 Mesa Rim Road, San Diego, CA 92121, U.S.A. • Tel: 1-858-875-8000 • Fax: 1-858-200-0729 • Email: info@aconlabs.com  
Please visit our website for details: <http://uscc.aconlabs.com>