



Early Detection Pregnancy Test

Detects Pregnancy up to 6 Days Early
Quick Results

Highly Reliable and >99% Accurate

- *Wide Tip*
- *Ergonomic Handle*

AVAILABLE FOR OEM OPPORTUNITIES

Offering customers an affordable and reliable testing option!

20 YEARS
20 Years
LATERAL FLOW EXPERIENCE

 **Highly Reliable**
AND ACCURATE

Detection of Pregnancy

Human chorionic gonadotropin (hCG) is a glycoprotein hormone produced by the developing placenta shortly after conception. hCG levels will increase throughout pregnancy. Specific hCG concentrations can be used to predict different stages of pregnancy.

General Information

The Distinct Early Detection Pregnancy Test is a rapid chromatographic immunoassay for the qualitative detection of human chorionic gonadotropin (hCG) in urine. The test is sensitive enough to detect pregnancy up to 6 days early.

- Detection: Human chorionic gonadotropin
- Specimen: Urine

• Test Time: 1 - 3 Minutes

- Shelf life: 24 months
- Storage Temp.: 2 - 30 °C / 36 - 86°F

Features




- Wide Absorbent Tip
- + or - Results Interpretation
- Ergonomic Thumb Grip
- Protected Results Window

Test Procedure and Interpretation



Urine Stream

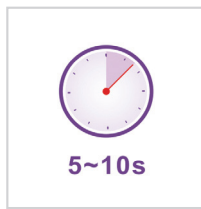

5~10s

OR



Collected Sample

Midstream Method

Hold the device by the Thumb Grip with the Absorbent Tip pointed downwards directly in the urine stream.

Keep the tip in the stream for 5 - 10 seconds.

Collected Sample Method

Collect a sample of your urine in a clean, dry container. Hold the device by the Thumb Grip with the Absorbent Tip pointed downwards into the collected urine sample.

Keep the tip submerged for 5-10 seconds.

'Pregnant' result may be seen as early as 1 minute. Wait the full 3 minutes to confirm a 'Not Pregnant' result.



Ordering Information

| PRODUCT NAME | CATALOGUE | FORMAT | SPECIMEN | PACKAGE |
|--------------------------------|------------|-----------|----------|--------------------------|
| Early Detection Pregnancy Test | L031-20192 | Midstream | Urine | 2 Tests/Kit; 3 Tests/Kit |



aconlabs.com

ACON Laboratories, Inc.
San Diego, CA 92121, U.S.A.
Tel: 1.858.875.8000
Fax: 1.858.200.0729
Email: info@aconlabs.com