



# Distinct Early Detection Digital Pregnancy Test Procedure

If you are testing before your expected period, use your first-morning urine, as it contains the most pregnancy hormone. If testing on the day of or any day after your expected period, you can test any time of the day.


1. Remove the test from the pouch and remove the Purple Cap. You can perform the test by either of two ways:

**Midstream Method:** Hold the device by the Thumb Grip with the Absorbent Tip pointed downwards directly in the urine stream. Keep the Absorbent Tip in the stream for 5 -10 seconds.


**Collected Sample Method:** Collect a sample of your urine in a clean, dry container. Hold the device by the Thumb Grip, with the Absorbent Tip pointed downwards, into the collected urine sample. Keep the tip submerged for 5-10 seconds.



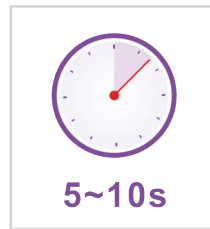

Urine Stream

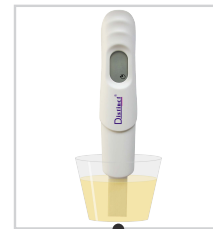

OR



Collected Sample

- After sampling, the Timer Icon  will be present on the Display Screen and start to flash after a few seconds. Immediately replace the Purple Cap on the Absorbent Tip. Lay the test on a flat surface. **Wait 3 minutes for the test to complete.** 
- Results will develop over the course of 3 minutes. A 'Pregnant' result may be seen as early as 1 minute after testing. You must wait 3 full minutes to confirm a 'Not Pregnant' result. Do not interpret the results after 10 minutes.



Pregnant



Pregnant



Pregnant



Not Pregnant



Invalid



Invalid



aconlabs.com

ACON Laboratories, Inc.  
San Diego, CA 92121, U.S.A.  
Tel: 1.800.838.9502  
Fax: 1.858.200.0729  
Email: info@aconlabs.com

