



Distinct Early Detection Pregnancy Test Frequently Asked Questions (FAQ)

Q: How does the test work?

A: The Distinct Early Detection Pregnancy Test detects a hormone (human Chorionic Gonadotropin - hCG) in your urine that your body produces during pregnancy. The amount of pregnancy hormone in your body increases as pregnancy progresses.

Q: How soon after I suspect that I am pregnant can I take the test?

A: You can use Distinct Early Detection Pregnancy Test up to 6 days before your missed period (5 days before the day of your expected period). If you test before your expected period and get a 'Not Pregnant' result, there is still a chance that you may be pregnant. See the chart below for results from clinical testing of Distinct Early Detection Pregnancy Test with early pregnancy samples.

Testing Early						
Number of days before the expected period	0	1	2	3	4	5
% of samples from pregnant women giving a 'pregnant' result	> 99%	> 99%	> 99%	97%	94%	75%

Q: Do I have to test with my first morning urine?

A: If you are testing before your expected period, we recommend that you test with your first morning urine as it is usually the most concentrated of the day and would have the most pregnancy hormone (hCG) in it. If you are testing on the day of, or after your missed period, you can test at any time of the day. Please avoid excessive fluid intake before testing.

Q: How accurate is the test?

A: A clinical evaluation was conducted comparing the Distinct Early Detection Pregnancy Test to a market leading pregnancy test. The results demonstrated >99% overall accuracy.

Q: How do I know the test was run properly?

A: The appearance of a blue vertical line in the Control Window indicates that the correct amount of urine was applied to the absorbent tip and the test was performed properly. If there is no blue vertical line in the control window and/or no blue '-' (minus) symbol in the Test Window the test was not performed properly and the result is invalid. Please repeat the test with a new pregnancy test. Make sure to apply urine to the absorbent tip for at least 5 seconds, but no more than 10 seconds.

Q: Can any conditions affect the result?

A: Medications containing hCG can affect the result.
Ectopic pregnancy and recent pregnancy, even if not carried to full term can cause misleading results.
If you are in, or approaching, menopause you may obtain a false pregnant result even though you are not pregnant.
If you get unexpected results discuss them with your doctor.



aconlabs.com

ACON Laboratories, Inc.
San Diego, CA 92121, U.S.A.
Tel: 1.800.838.9502
Fax: 1.858.200.0729
Email: info@aconlabs.com

