



Distinct Early Detection Digital Pregnancy Test Frequently Asked Questions (FAQ)

Obtain a result as early as possible. The test is designed to detect pregnancy as early as possible, even before your missed period. The test is accurate and reliable, providing a clear result in minutes.

Obtain a result as early as possible. The test is designed to detect pregnancy as early as possible, even before your missed period. The test is accurate and reliable, providing a clear result in minutes.

Testing Early						
Number of days before the expected period	0	1	2	3	4	5
Percentage of samples giving a "pregnant" result	> 99%	> 99%	99%	94%	89%	72%

Q: Do I have to test with my first morning urine?

A: If you are testing before your expected period, we recommend that you test with your first morning urine as it is usually the most concentrated of the day and would have the most pregnancy hormone (hCG) in it. If you are testing on the day of, or after your missed period, you can test at any time of the day. Please avoid excessive fluid intake before testing.

Q: How accurate is the test?

A: A clinical evaluation was conducted comparing the Distinct Early Detection Pregnancy Test to a market leading pregnancy test. The results demonstrated > 99% overall accuracy.

Q: Can any conditions affect the result?

A: Medications containing hCG can affect the result.

Ectopic pregnancy and recent pregnancy, even if not carried to full term can cause misleading results.

If you are in, or approaching, menopause you may obtain a false pregnant result even though you are not pregnant.

If you get unexpected results discuss them with your doctor.



aconlabs.com

ACON Laboratories, Inc.
San Diego, CA 92121, U.S.A.
Tel: 1.800.838.9502
Fax: 1.858.200.0729
Email: info@aconlabs.com

