



HealthyMe® Early Detection Pregnancy Test

Results up to
6 Days Early*
Over 99% Accurate



2 Tests

* See back of box

Early Detection Pregnancy Test

Detects Pregnancy up to 6 Days Early
Quick Results

Highly Reliable and >99% Accurate

- Wide Tip
- Ergonomic Handle

AVAILABLE FOR OEM OPPORTUNITIES

Offering customers an affordable and reliable testing option!



Detection of Pregnancy

Human chorionic gonadotropin (hCG) is a glycoprotein hormone produced by the developing placenta shortly after conception. hCG levels will increase throughout pregnancy. Specific hCG concentrations can be used to predict different stages of pregnancy.

General Information

The HealthyMe Early Detection Pregnancy Test is a rapid chromatographic immunoassay for the qualitative detection of human chorionic gonadotropin (hCG) in urine. The test is sensitive enough to detect pregnancy up to 6 days early.

- Detection: Human chorionic gonadotropin
- Specimen: Urine

- Test Time: 1 - 3 Minutes

- Shelf life: 24 months
- Storage Temp.: 2 - 30 °C / 36 - 86°F

Features




- Wide Absorbent Tip
- + or - Results Interpretation
- Ergonomic Thumb Grip
- Protected Results Window

Test Procedure and Interpretation



Urine Stream

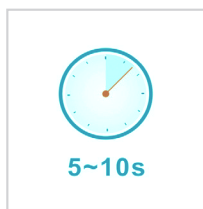

OR



Collected Sample

Midstream Method

Hold the device by the Thumb Grip with the Absorbent Tip pointed downwards directly in the urine stream.


Keep the tip in the stream for 5 - 10 seconds.

Collected Sample Method

Collect a sample of your urine in a clean, dry container. Hold the device by the Thumb Grip with the Absorbent Tip pointed downwards into the collected urine sample.

Keep the tip submerged for 5-10 seconds.

'Pregnant' result may be seen as early as 1 minute. Wait the full 3 minutes to confirm a 'Not Pregnant' result.

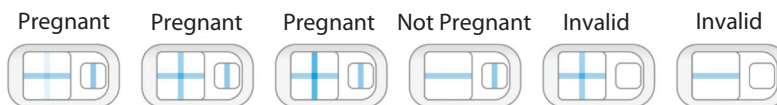

Ordering Information

FDA Cleared for OTC

PRODUCT NAME	CATALOGUE	FORMAT	SPECIMEN	PACKAGE
Early Detection Pregnancy Test	L031-20198	Midstream	Urine	2 Tests/Kit; 3 Tests/Kit



aconlabs.com

ACON Laboratories, Inc.
San Diego, CA 92121, U.S.A.
Tel: 1.858.875.8000
Fax: 1.858.200.0729
Email: info@aconlabs.com